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Product description

Bowie & Dick (B&D) test is intended for checking the air removal effect of Steam Sterilizer with pre-vacuum system. It can be used for daily monitoring, projection of the operation procedure, effects measurement after commissioning, performance measurement after the maintenance of the sterilizer, etc. It consists of a B&D image packed in porous paper. The image pattern changes from light yellow to dark brown or black under certain conditions of pressure and humidity.

Instructions for use

1. Place the Bowie Dick test pack inside the chamber (empty), horizontally if put in a tray or vertically if shelter, in the lower section and close to the door.
2. Perform a steam sterilization cycle at 134 °C for 3.5 min.
3. Open the chamber door.
4. Remove the Bowie Dick test pack, open it, remove the pattern in the middle of the package and watch it.

Obs: This product does not need to be used in conjunction with the standard test kit described in the Disinfection Technical Specifications.

Result determination

Qualified: The test pattern changes from light yellow to dark brown or black, that is, the color of the central part and the end is the same. The B&D test is qualified, indicating that air has been completely removed, the sterilizer works well and does not leak.

Failed: The test pattern does not change color or is not completely discolored. Generally, the central part is lighter than the end. The B&D test does not match, indicating that the air has not been completely removed or leaked.

Attention

The test shall be performed with the chamber empty, conducted at a temperature of 134 °C for a period of not more than 4 minutes, prior to the first daily sterilization.

This product cannot be used as a test to detect the effect of steam sterilization.

Storage conditions

Prohibited contact with acids and alkalis.

Relative humidity: <50%.

Store in a dark place, protected from ultraviolet light, fluorescent light, and sunlight.

Shelf life

36 months.

The production date and the batch number can be found on the packaging.